# English

## L1 Breathing Tube User Manual

0123

- ◆ **Before using the Product, please read this user manual through carefully and completely.**

### ⚠ Warnings and Precautions

- Please keep this user manual for future reference.
- Always touch the tube at the connector, when connecting or disconnecting the therapy device, humidifier and mask.
- The tube must not be heated (>60°C / >140°F).
- The tube may be used by one patient only.
- After washing, the tube must not be dried using a (microwave) oven, a hairdryer or other appliances. See cleaning instructions.
- The interior of the tube must not be cleaned with brushes or other objects, as this can damage the tube.
- Do not disassemble the tube or change it.
- Do not insert or drop any object in the openings of the tube as it can damage the tube.
- Do not place objects on the tube as it can damage the tube.
- If the tube is damaged (holes, creases, tears, etc.) or no longer functional, do not try to repair it. Replace the tube immediately. Label the old tube so that it is not used by another patient before its final disposal.
- Any serious incidents that occur in relation to this device should be reported to BMC and to the competent authority in your country.

### Important Tip!

- ◆ Read and understand the entire user manual before operating this system. If you have any questions concerning the use of this system, contact your home care provider or health care professional.

### Intended Use

The tube is intended to transport breathing gasses from the flow generator or humidifier to the patient.

The tube is indicated for non-invasive respiratory therapy in the home, hospital or sleep-lab setting by a single adult patient. It can also be used in conjunction with supplemental Oxygen.

#### Note:

- ◆ The tube is intended for use with respiratory therapy devices and humidifiers with conical ISO connectors (ISO 5356). L1 Breathing Tube which is intended to engage with 22 mm male conical Connectors complying with ISO 5356-1 has plain ends (connectors) complying with 5.3.2 of ISO 5367-2014.

### Intended Clinical Benefits to Patients

1. Improve patients' quality of sleep.
2. Improve patients' compliance with therapy.

### Construction of the L1 Breathing Tube

The L1 Breathing Tube is made up of double Connectors and Threaded Body. Double Connectors and Threaded Body cannot be separable.

### Cleaning

#### ⚠ Warnings:

- ◆ Do not use harsh soaps, detergents, solvents or alcohols to clean the tube.
- ◆ Do not clean the tube in a dishwasher.

#### Note:

- ◆ The tube is recommended to be cleaned once a day or after each use.

### Preparation before cleaning

1. Hold the tube connector and disconnect the tube from the devices.
2. Preparation tools: soft bristle brush, sterile water (five litres per tube), mild liquid detergent (for example: dilute the soap with clean water to make 2% soapy water).

### Cleaning Instructions

1. Soak the tube in 2% soapy water diluted with potable water at 35°C to 40°C for 10 minutes.
2. Then clean the inside and outside surfaces with a soft bristle brush 5 times.
3. Finally, rinse the inside and outside surfaces with sterile water for 3 times, 1 minute each time.
4. Gently tap the tube to remove excess moisture from the connector ports. Hang the tube so that both ends of the opening are facing the ground. Let the tube naturally dry, out of direct sunlight.

### Disinfection

This product is intended for single patient use only. Disinfection of the tube is generally not necessary if the cleaning instructions are strictly followed. When the product is contaminated, clean it according to the cleaning instructions first, and then disinfect it according to the following steps.

1. Immerse the tube in 0.55% O-phthalaldehyde for 12 minutes. Ensure that the solution contacts all surfaces.
2. After immersing, rinse the tube thoroughly under potable water for 5 minutes.
3. Tap gently to remove excess moisture from the connector port.
4. Suspend the tube with the openings at both ends facing the floor.
5. Air-dry the tube, and avoid direct sunlight.

### ⚠ Warning

- ◆ Sterilization of the product is unnecessary.

#### Notes:

- ◆ The tube can be cleaned up to 180 times and be disinfected up to 100 times.
- ◆ Disinfectants generally damage materials and result in reduced life of components. The instructions for use of disinfectants should be noted. Please select the appropriate disinfectant, and follow the disinfectant manufacturer's recommendations.
- ◆ After disinfection, check the disinfected tube for any signs of damage (such as rupture or broken parts). Replace the damaged tube immediately. There may be slight discoloration of the tube after disinfection, which is normal.
- ◆ Be sure to rinse the tube thoroughly with water after disinfection to prevent residual disinfectant from damaging the skin and respiratory tract or causing allergic reactions.

### Product Specifications

Tube	Length	Internal diameter	Compliance standards	Resistance to flow at rated flow
D 19 / D 15	1,8 m	19 mm / 15 mm	60 hPa: < 5 ml/hPa	30 l/min: < 0.06 hPa/l/min

Operating conditions: Ambient Temperature: 5°C to 35°C (41°F to 95°F)  
Relative humidity: 0% to 93%  
Storage and transport conditions:  
Storage temperature: -25°C to 70°C (-13°F to 158°F)  
Relative humidity: 0% to 93%  
The tube should be transported and stored in the original packaging and protected from dust and exposure to direct sunlight.

### Disposal

This product and packaging do not contain any hazardous substances. After use, the product should not be discarded at will and must be disposed of as medical waste, otherwise it may cause the spread of disease.

### Service Life

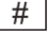
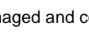
Under normal maintenance and usage, the expected service life of the tube is 6 months. After 6 months, if the problem (holes, creases, tears) is still not found after inspection, the tube can still be used.

### Warranty

It is warranted that the tube shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer. To exercise the rights under this warranty, contact the local authorized dealers. The tube is a consumable and must be checked before first use otherwise warranty claims are excluded. Warranty expires, if:

- ◆ Interventions, modifications, extensions, calibration, repairs and maintenance are carried out by persons who are not authorized by the manufacturer.
- ◆ Accessories and spare parts from other manufacturers not approved by BMC are used with the tube.
- ◆ The product is used other than as described in the user manual.
- ◆ Wearing parts are worn.
- ◆ Product damages are caused by force majeure (e.g.by lightning).
- ◆ The hygiene and cleaning instructions described in the user manual have not been complied with.

### System and Packaging

	Caution		Logo of BMC Medical Co., Ltd.
	Batch code		Authorized Representative in the European Community
	Catalogue number		CE marking
	Model Number		Made in China
	Medical device		Manufacturer
	Unique device identifier		Use-by date
	Importer		Temperature limit
	Single patient multiple use		Humidity limitation
	Consult instructions for use		Atmospheric pressure limitation
	Do not use if package is damaged and consult instructions for use		

**MANUFACTURER:**  
**BMC MEDICAL CO., LTD.**  
Room 110 Tower A Fengyu Building, No. 115 Fucheng Road, Haidian, 100036 Beijing, PEOPLES REPUBLIC OF CHINA  
URL: en.bmc-medical.com  
E-mail: intl@bmc-medical.com  
Tel: +86-10-51663880  
Fax: +86-10-51663880 Ext. 810

**EU AUTHORISED REPRESENTATIVE:**  
**Shanghai International Holding Corp. GmbH (Europe)**  
EiffestraÙe 80, 20537 Hamburg, Germany  
Tel: 0049-40-2513175  
Fax: 0049-40-255726

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